

**SCAN OPTICS**

**SO-161-R  
MICROSCOPE**

**USER MANUAL**

## CONTENTS

<i>CONTENTS</i>	2
<i>LIST OF FIGURES</i>	2
<i>INTRODUCTION</i>	3
<i>INTRODUCTION</i>	3
<i>PARTS LIST</i>	5
<i>ASSEMBLY INSTRUCTIONS</i>	6
TABLE MOUNTING	6
MOUNTING TO THE SO-331 FLOOR STAND	7
BATTERY CONNECTION, OPERATION, MAINTENANCE AND SAFETY	9
<i>USING THE MICROSCOPE</i>	10
ARTICULATED ARM	10
POSITIONING THE MICROSCOPE	11
POWER SUPPLY CONTROLS	11
LAMP REPLACEMENT	12
STERILISATION AND CLEANING	13
CARE OF THE OPTICAL HEAD	14
ADJUSTING FOCUS FRICTION	15
MOULD PELLET REPLACEMENT	15
<i>SPECIFICATIONS</i>	17

## LIST OF FIGURES

Figure 1: SO-161-R Microscope	3
Figure 2: SO-161-R Microscope shown with SO-331 Floor stand	4
Figure 3: Assembling the SO-161-R Microscope	7
Figure 4: Assembling the SO-161-R Microscope on SO-331 Floor stand	8
Figure 5: SO-161-R Power supply bottom panel	9
Figure 6: Articulated arm features	10
Figure 7: SO-161-R Power supply top panel	11
Figure 8: Changing the lamp	12
Figure 9: Sterilisable covers	13
Figure 10: Replacing the mould pellet	15


## INTRODUCTION

Please read the following information carefully before installing and using the Scan Optics SO-161-R Ophthalmic microscope. Scan Optics is responsible for the safety, reliability and performance of the equipment only if it is used in accordance with these instructions.


This microscope is designed for use by a certified practitioner, for magnified observation of patients, and for use in an operating theatre as an observation aid during surgery. Environmental storage and packing conditions of 60-95% relative humidity and 10-40 °C, are recommended for this product.

No parts or accessories supplied with this microscope are supplied in a sterile condition.

Apart from those instructions within this manual, there are no user-serviceable parts in this microscope. Scan Optics will retain the discretion to advise whether any repairs may be taken out by external qualified technical personnel, or whether part(s) of the microscope must be returned to the manufacturer's premises for service or repairs to be carried out under warranty or otherwise. Where appropriately qualified technical personnel are identified by a user, and ratified by Scan Optics, then Scan Optics will make available on request any information that will assist in repairing the equipment.

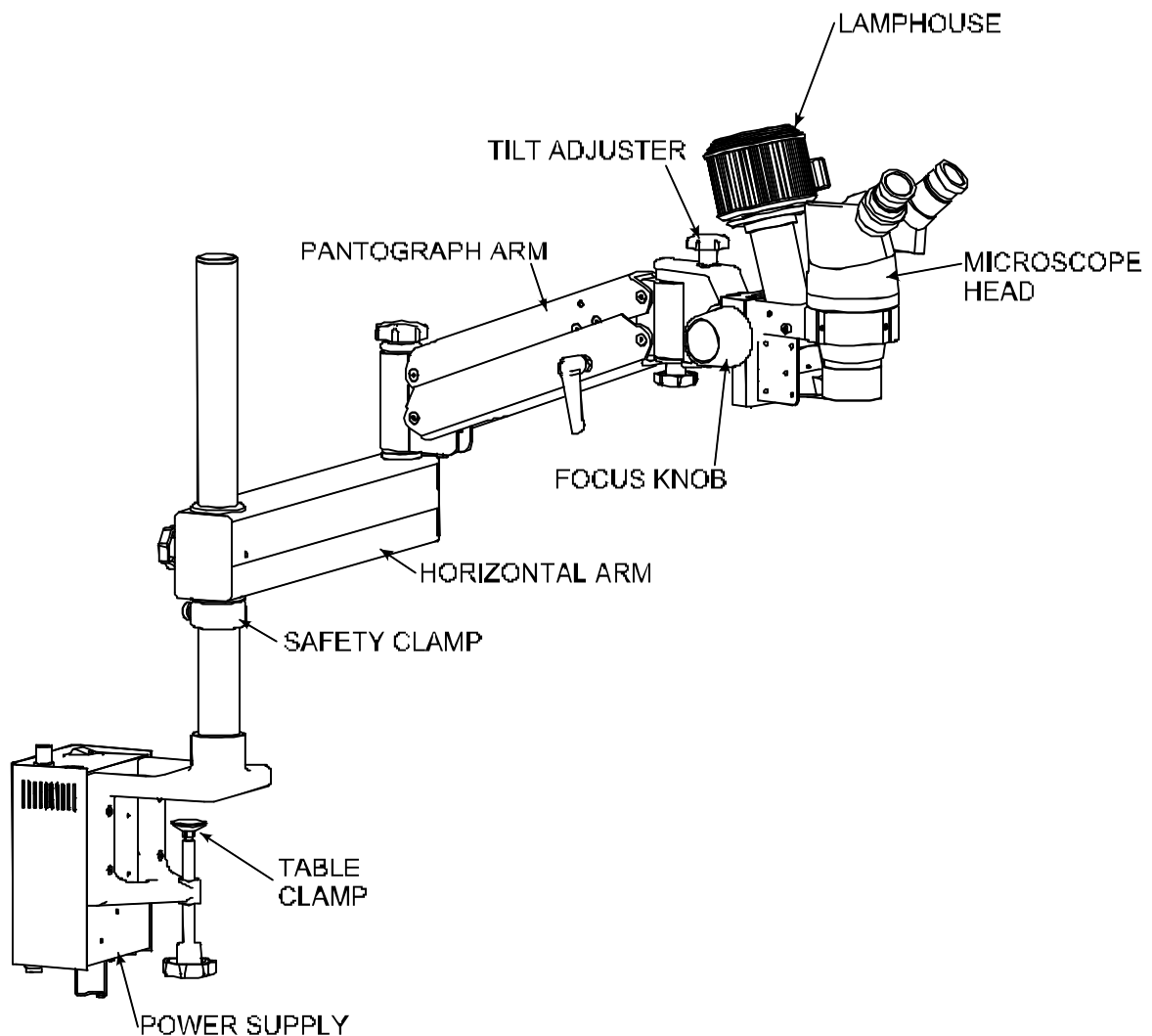

**Figure 1: SO-161-R Microscope**

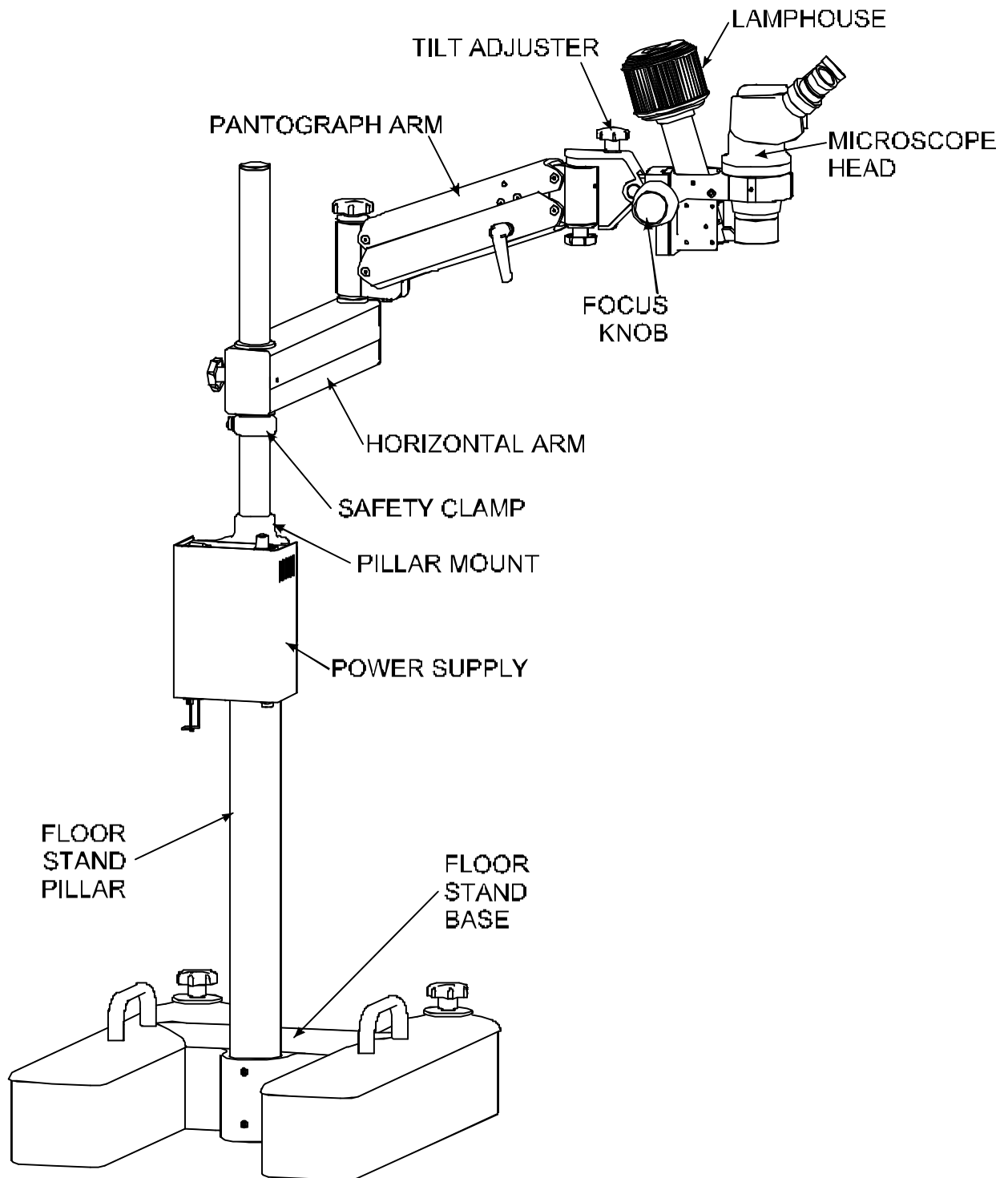

**Figure 2: SO-161-R Microscope shown with SO-331 Floor stand**

## PARTS LIST

### MAIN ASSEMBLIES

Power supply assembly	includes power supply mounted on G-clamp/pillar assembly and safety clamp
Articulated Arm Assembly	includes horizontal arm and adjustable pantograph arm
Microscope Assembly	includes tilt adjuster, microscope head, guide handle, lamphouse, universal arm and cable

### CABLES

Mains Cable	1
12V dc Supply (Battery) Cable	1

### OTHER

Focus Knob Sterilisable Covers	2
Eyepieces	2

### TOOL KIT

Spare lamp	1
Socket keys	set 7: 1.5mm, 2mm, 2.5mm, 3mm, 4mm, 5mm, and 6mm
Lens Cloth	1

### OPTIONAL ACCESSORIES

### SCAN OPTICS PART NUMBER

Floor stand (heavyweight)	SO-331*
Floor stand (lightweight)	SO-311
Binocular assistant microscope	SO-1420
Pupillary distance setter	SO-1510
Side handle	
Guide handle	

\*Note that in SO-161-R microscopes supplied with the SO-331 floor stand, the power supply assembly is mounted on a pillar fitting, not a G-clamp. Refer to this manual for details.

## ASSEMBLY INSTRUCTIONS

### TABLE MOUNTING


1. Remove the power supply assembly from the carton, and fix it firmly to the table using the G-clamp knob. Use hand tightening only to fix the assembly in place – use of a spanner or pliers to tighten the screw may strip the thread or produce excessive deflection in the clamp.
2. Move the safety clamp up the pillar to the desired height of the fixed arm, and lock it in place with the thumbscrew.
3. Remove the arm assembly from the carton and loosen the elbow lock knob to allow the pantograph arm to rotate about the end of the fixed arm. Then place the arm on the pillar so that it rests on the safety clamp.
4. Remove the head assembly from the carton and place the pin into the collet at the end of the pantograph arm. Tighten the wrist lock knob under the collet to secure the head assembly in place.
5. Connect the lamphouse cable to the socket located on the top of the power supply box.
6. Connect the mains cable to the IEC socket located on the under-side of the power supply box. Use the cable-retaining clip to secure the plug in place. Connect the other end of the mains cable to a reliable EARTHED mains power supply.
7. Check the setting of the voltage selector located on the under-side of the power supply box. If necessary, change the setting to 120 or 240V as required, using a flat-head screwdriver or coin. Severe damage to the power supply may result if this is incorrectly set.
8. Switch on the mains power supply at the mains socket. When the ON/OFF switch is selected to ON, the power supply indicator on the switch will light up.
9. Remove the microscope eyepiece blanks from the head assembly and insert the eyepieces. Retain the eyepiece blanks in the eyepiece packaging for when the microscope is repacked. Do not discard the eyepiece blanks.
10. When focusing the microscope, look with the right eye through the right eyepiece first and use the focus knob or foot control to bring a target into focus at the preferred magnification. Then look through the left eyepiece with the left eye and adjust the dial to compensate for any relative refractive error between the eyes.
11. Remove two focus knob covers from the bag inside the carton. Push them into position on to the focus knobs on either side of the microscope.

### CAUTION:

- Scan Optics recommend that ALL packaging materials be retained by the user in a safe place, in case of future transit needs.
- All covers are intended to be sterilised before any operating procedures.
- **The mains power supply must have a protective earth conductor.** If there is no earth conductor, or if the integrity of the earth conductor arrangement is in doubt, the equipment must be operated from a 12Vdc power source.

**MOUNTING TO THE SO-331 FLOOR STAND**

1. Carefully lift the floor stand base out of the carton using the handles that are fixed to the top. The floor stand base is extremely heavy and should only ever be lifted with the assistance of another person. Place the floor stand base on the floor. Remove the pillar from the carton and place it in the floor stand base. Tighten the two grub screws using the 5mm socket key found in the tool kit to fix the pillar assembly in place.
2. To fix the floor stand base, screw the knobs down until the stops are resting evenly on the floor, then screw the knurled discs down on to the floor stand base to lock the stops. To unlock the floor stand, screw the knurled discs back up, and then unscrew the knobs until the wheels can move freely.
3. Place the pillar fitting/power supply assembly on the pillar and tighten the two grub screws to fix the assembly in place.
4. Continue assembling the microscope from step 2 of the instructions for table mounting.

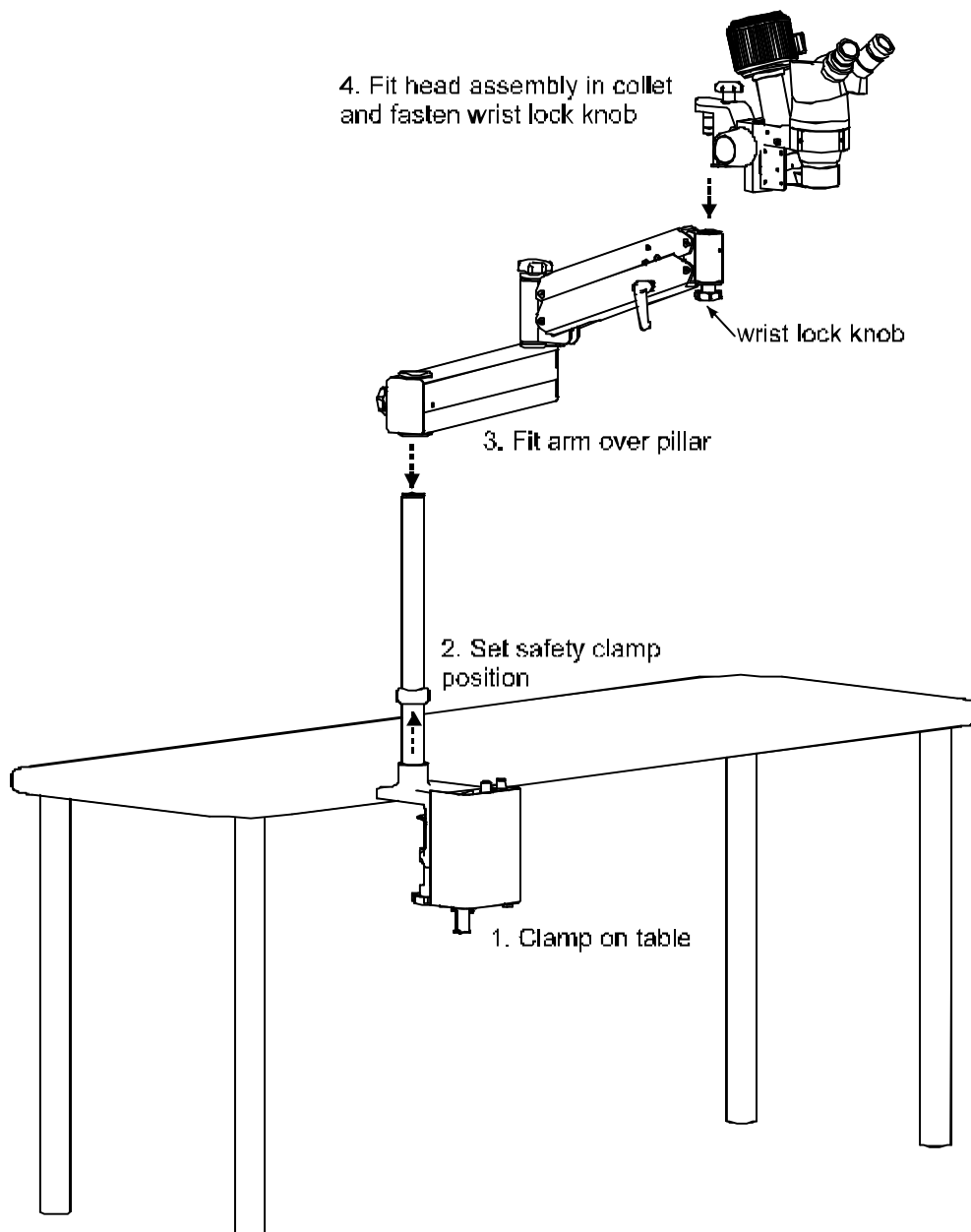**Figure 3: Assembling the SO-161-R Microscope**


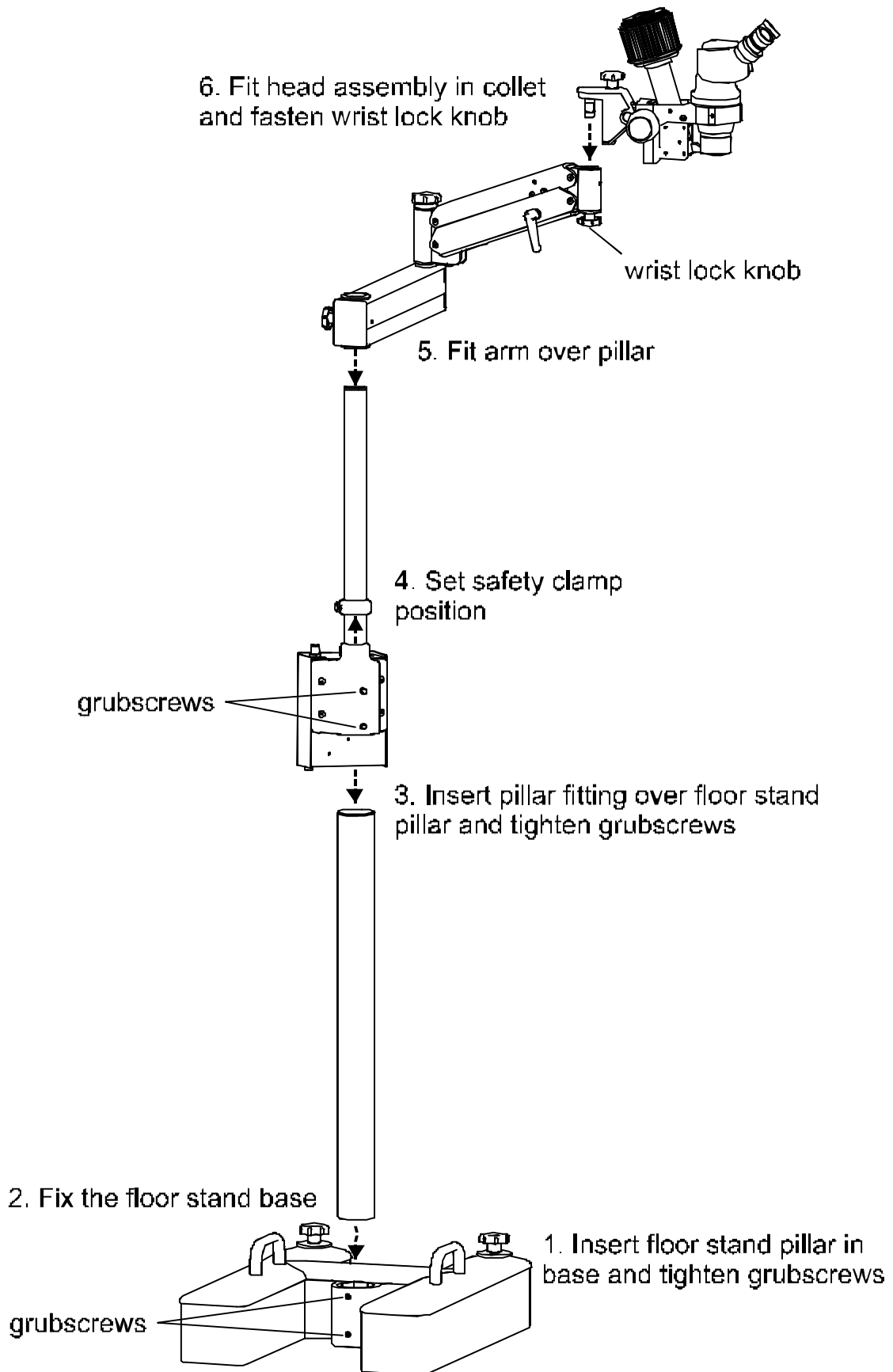


Figure 4: Assembling the SO-161-R Microscope on SO-331 Floor stand



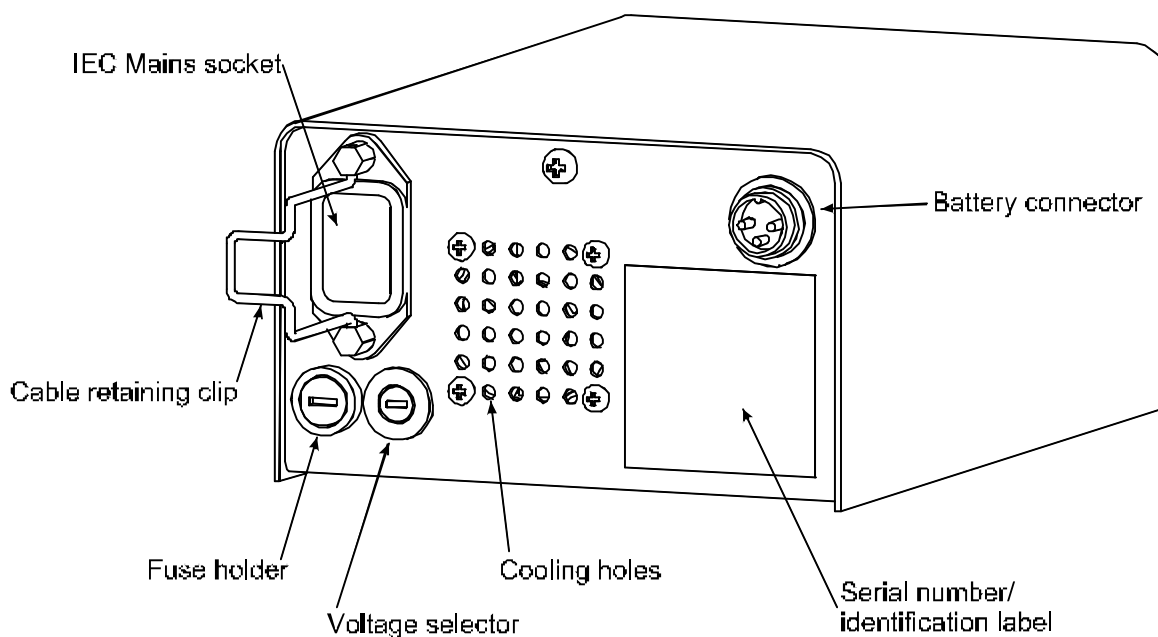

**Figure 5: SO-161-R Power supply bottom panel**

#### **MOUNTING TO THE SO-311 FLOOR STAND**

1. Assemble the floor stand according to the instructions provided with it.
2. Clamp the power supply assembly to the mounting plate at the top of the floor stand.
3. Continue assembling the microscope from step 2 of the instructions for table mounting.

#### **CAUTION:**

- Scan Optics advise caution when using the SO-311 floor stand with this microscope. To prevent over-balancing, use sandbags or a similar method to weigh down the floor stand legs. Try to use the fixed arm and pantograph arm in a compact orientation to minimize the moment arm acting on the floor stand.

#### **BATTERY CONNECTION, OPERATION, MAINTENANCE AND SAFETY**

Scan Optics recommend the use of sealed rechargeable lead-acid 12V batteries such as Scan Optics Cat Nos. SO-251 (small, light, lower (7Ah) capacity) and SO-9210 (large, heavy, higher (35Ah) capacity). These batteries are **maintenance-free** and can be operated, charged or stored in any position without leakage.

1. If the power supply is to be connected to a 12-volt dc supply, remove the battery cable from the carton and connect the cable to the 3-pin plug on the underside of the power supply.
2. Connect the red battery clip to the positive battery terminal, and the black clip to the negative battery terminal. The power supply will not operate if the terminals are reversed

#### **CAUTION:**


- The 12-volt supply must be direct current. The power supply will not operate with 12 volts alternating current.
- Do not charge these batteries in a sealed container.
- Avoid short-circuiting batteries.
- Old lead-acid batteries of any type must be disposed of correctly. It is recommended that they are recycled by an appropriate establishment who recycle car batteries. Lead acid batteries should not be disposed of with ordinary household or medical waste, as lead poisoning or acid trauma may result.
- Where battery backup is used, Scan Optics recommends a periodical check of the battery to ensure it is charged and functional.

## USING THE MICROSCOPE

### ARTICULATED ARM

The articulated arm includes a number of features that enable the microscope to be adjusted in almost any position.

1. The horizontal arm may rotate about the vertical pillar by unlocking the pillar lock knob. To prevent the arm from rotating, simply lock the knob.
2. The pantograph (moving) arm may rotate about the end of the horizontal arm by unlocking the elbow lock knob. To prevent the arm from rotating, simply lock the knob.
3. The amount of force required to move the pantograph arm up and down may be adjusted by rotating the screw located at the top of the elbow joint. To adjust the gas spring force, first move the pantograph arm down to reveal the screw through the slot in the top of the arm. Use the socket key located in the toolbox to rotate the screw. To decrease the amount of force required to move the arm, rotate the screw clockwise. To increase the amount of force required to move the arm, rotate the screw anticlockwise. If the screw is adjusted to its limit, rotate the screw half a turn in the opposite direction to ensure the arm continues to operate smoothly.
4. The amount of friction in the pantograph arm may be adjusted by rotating the handle located on the side of the pantograph arm. The arm may be locked in any position or alternatively, the amount of friction can be set so that the arm is stable but will move when a force is applied.
5. The wrist lock knob located on the under-side of the end of the pantograph arm is used to lock the microscope assembly into place. To release the microscope assembly, simply unscrew the knob and lift the assembly carefully out of the collet.
6. The microscope head may be tilted up by some 5° and down by 45°, by rotating the tilt adjustment knob. To tilt the head down, rotate the knob in the clockwise direction. To tilt the head up, rotate the knob anticlockwise.

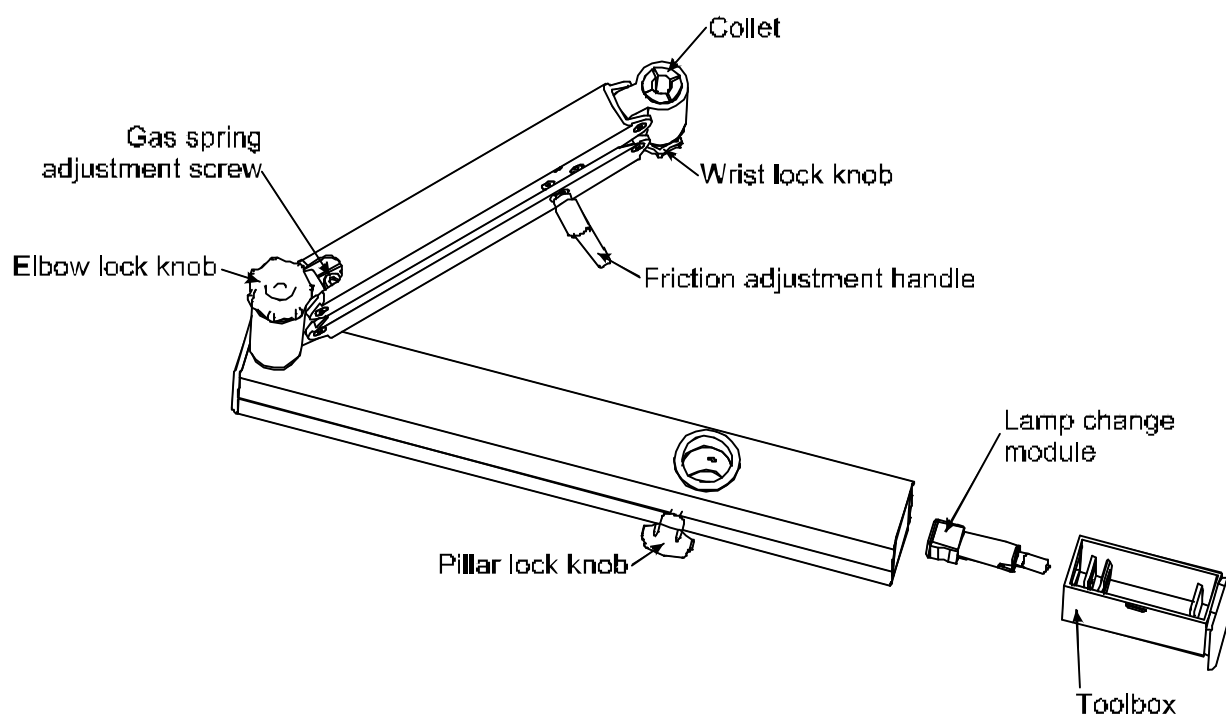

**Figure 6: Articulated arm features**


### POSITIONING THE MICROSCOPE

Note that the equipment must be located more than 25 cm away from any medical gas system or disinfection or degreasing system containing flammable vapour. The power supply must also be protected from liquid splashes and spills.

1. Set the instrument approximately in position by swinging the elbow as required.
2. Set the focus adjustment to the midway position by rotating the focus knob appropriately.
3. Adjust the height of the microscope by moving the arm vertically so that the work area is approximately in focus. Tighten the pillar and elbow lock knobs.
4. Check the eyepiece setting to ensure clear vision with each eye separately, and set the pupillary distance. Note that the working distance is increased by rotating both eyepieces in a clockwise direction, and reduced by rotating both eyepieces counter-clockwise.
5. The microscope can now be swung aside ready for use with a patient.
6. Swing the microscope over the patient
7. Tighten the pillar lock knob until the microscope is prevented from moving freely, but is still able to be moved when required. The friction of the arm and elbow knobs should be adjusted so that the movement feels uniform in all directions.
8. To swing the microscope out of the way, unlock the pillar lock knob and swing the microscope about the vertical pillar. It will remain in focus when returned to the work area.

### POWER SUPPLY CONTROLS

Switch the microscope on using the mains switch – the LED mounted in the switch will glow when the microscope is switched on. Use the manual light intensity adjustment to obtain a suitable setting. Note that lamp life will be extended if a lower light intensity setting is used.

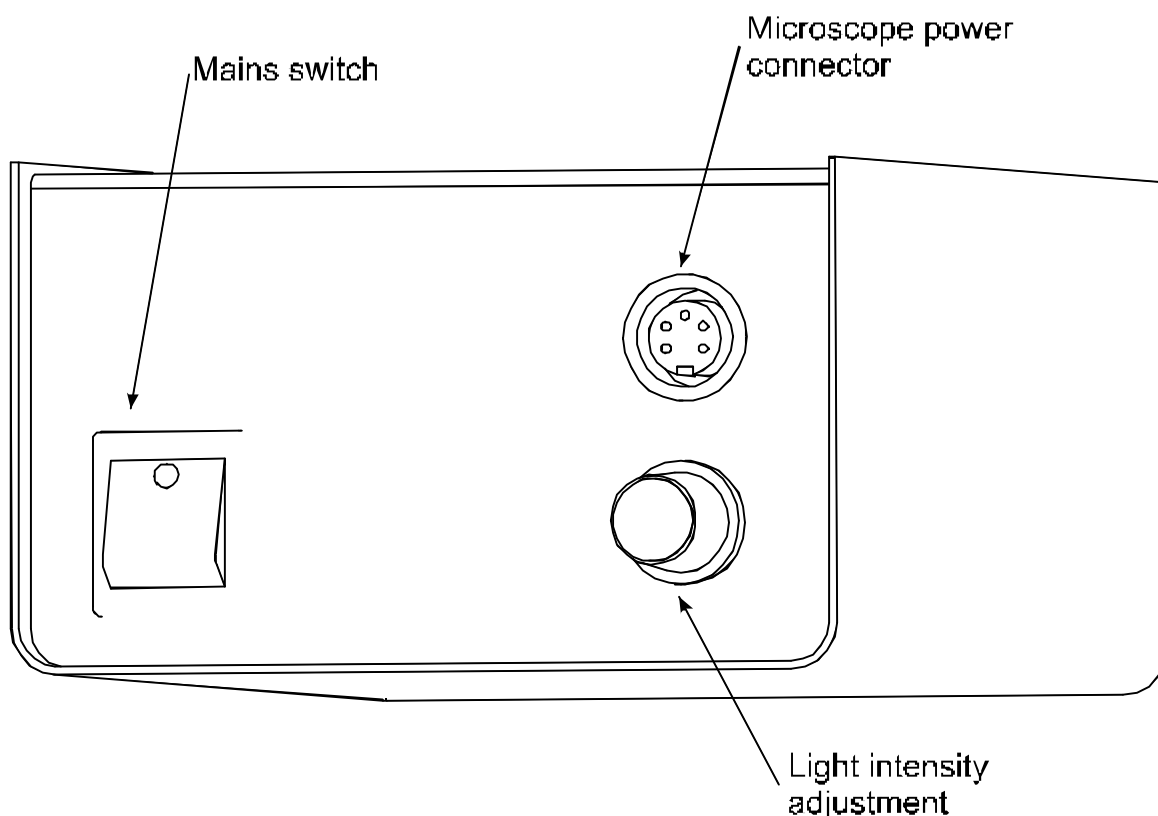

**Figure 7: SO-161-R Power supply top panel**

### LAMPHOUSE AND LAMP

1. The lamp house on the SO-161-R ophthalmic microscope produces a coaxial light that provides a red fundus reflex from the back of the eye. The intensity of the light may be controlled from the top panel of the power supply. The lamp supplied has a rated average life of 50 hours, but the actual life of the lamp will depend on the intensity setting normally used. The highest setting will increase light output but will reduce lamp life. Conversely, running the lamp at the lowest setting will produce a lamp life of greater than 50 hours.
2. It is strongly recommended that the lamp be replaced as a routine maintenance task, to reduce the possibility of failure during surgery.

### LAMP REPLACEMENT

1. Remove the toolbox from the horizontal arm by pulling it out horizontally. The spare lamp module is located at the end of the toolbox, and may be removed by pulling out horizontally.
2. Pull out the old lamp module from the side of the lamp house by gripping the rectangular block located beneath the cable gland at the back of the lamp house. Place the replacement module in the lamp house so that the pins engage in the sockets and 'click' into place.
3. When the old lamp has cooled down, it may be replaced by gripping the lamp holder at the sides where the rectangular cutout is located, and pulling the lamp out. Do not attempt to pull the lamp out by gripping the glass capsule. Push the new lamp into the socket so that the lamp flange sits flush on the front of the locating tube. Ensure that the new lamp is free of grease or fingerprints before replacement. Use of protective gloves during this procedure may be useful. Any grease marks should be removed with a solvent such as methanol to avoid reducing lamp life. This assembly is now ready for use as a new quick-change lamp module and may be replaced in the toolbox

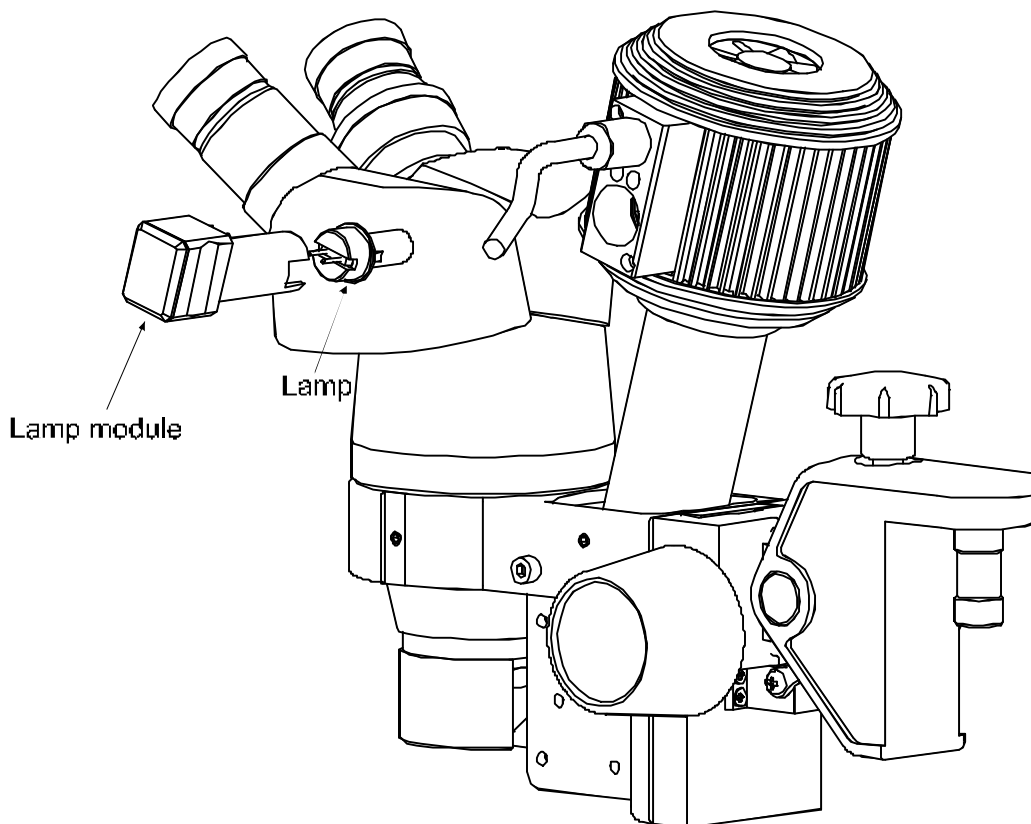



Figure 8: Changing the lamp

**STERILISATION AND CLEANING**

1. The removable covers may be sterilised by boiling, autoclaving, chemical sterilisation or gas sterilisation.
2. One set of covers can be sterilised while the other is in use.
3. The anodised and plated metal components can be wiped with any of the normal disinfectants.
4. The plastic parts and the paintwork of the microscope assembly and the power supply may be affected by organic solvents. Do not autoclave or wipe with organic solvents such as ether, xylene or alcohol. To clean, use water-based solvents only.
5. Focus covers are used to cover focus knobs only
6. Zoom covers are used to cover the optional SO-1420 binocular assistant microscope focusing knobs and SO-1510 PD adjuster knob
7. Guide handle covers are used for the optional guide handle and side handles

**CAUTION:**

- National authorities may require the use of specific sterilisation or disinfection methods.

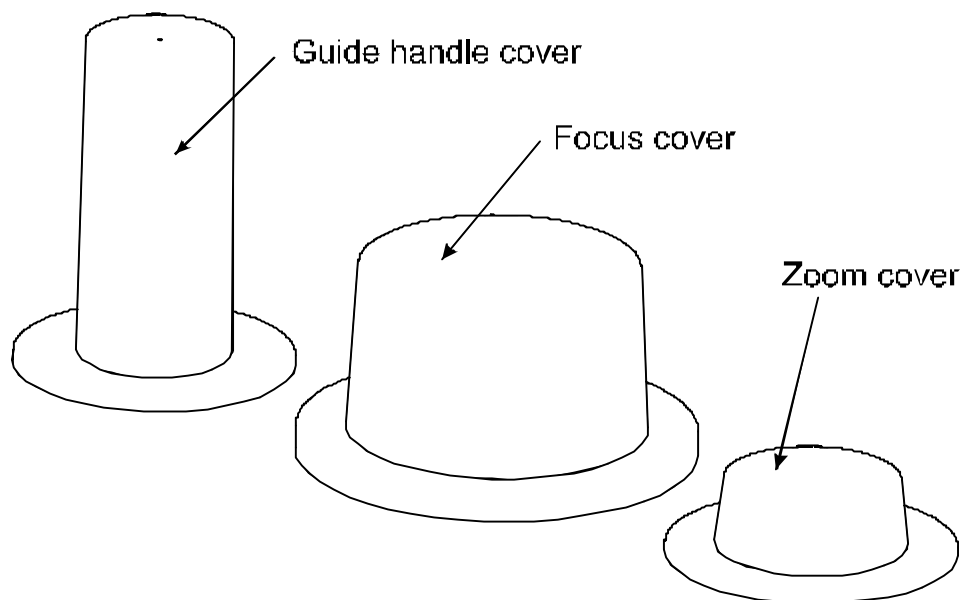

**Figure 9: Sterilisable covers**

## CARE AND MAINTENANCE

### CARE OF THE OPTICAL HEAD

1. *Cleaning the optical components.*  
The eyepieces should be checked for cleanliness each time the instrument is used. Surface dust should be removed with a clean, soft brush. Fingerprints or grease may be removed by lightly wiping with a cotton cloth or lens tissue moistened with a 70:30 mixture of absolute alcohol (either ethanol or methanol) and ether. **Do not use acetone as it may damage the surface coating.**
2. *Cleaning the plastic parts and paintwork.*  
Use water based cleaners only.  
**Do not use any organic solvent such as alcohol, ether or xylene.**
3. *Protection against mould.*  
In hot and humid climates, it is common for mould to grow on optical surfaces. Cleaning and repairing the damage can be expensive and inconvenient. To minimise the risk of mould forming, do not leave the instrument without either eyepieces or eyepiece blanks inserted and always store the optical head in a sealed bag containing silica gel desiccant. In tropical climates, routine annual maintenance of the optical head is recommended.
4. *Do not dismantle.*  
With the exception of the anti-mould pellet, no parts inside the optical head of the instrument can be serviced by the user. Attempts to dismantle the optical head or prism cover will make any warranty void.

### ADJUSTING FOCUS FRICTION

The focusing system should allow the microscope head to be lowered and raised smoothly and easily, but should not allow the head to fall under its own weight. To adjust the friction, simply hold the plain (left) focus control knob and rotate the right focus control knob (marked "tension") until the desired friction is achieved. To increase the friction rotate the knob clockwise, to decrease the friction rotate the knob anti-clockwise.

### MOULD PELLETT REPLACEMENT

Scan Optics microscopes are fitted with mould protection which lasts for approximately 3 years, however, this is dependent on the storage conditions and humidity of the local environment. In extreme tropical climates, it may be necessary to change the mould protection as frequently as every year.

A guide to when the mould protection should be changed is placed on a sticker on the front of the microscope head. However, this is indicative only and users should be guided by their own experience and knowledge of local conditions. As a general precaution, always store the microscope head in a protective bag when not in use, and replace the eyepieces with the protective eyepiece caps.

To replace the mould pellet:

1. Loosen the grub screws holding the head in the bonder arm.
2. Lift the head out of the arm and remove the prism protector.
3. Carefully unscrew the objective lens, taking care not to damage the lamp house prism.
4. Use a Philips head screwdriver to remove the mould pellet. Push the end into the hole and lever off the surface.
5. Place the new pellet on the end of the screwdriver, remove the protective cover and fix to the same location where the old pellet was removed.
6. Screw the objective lens back on and replace the prism protector.
7. Replace the head in the bonder arm ensuring that the head is aligned to the front of the arm and secure by tightening the grub screws.

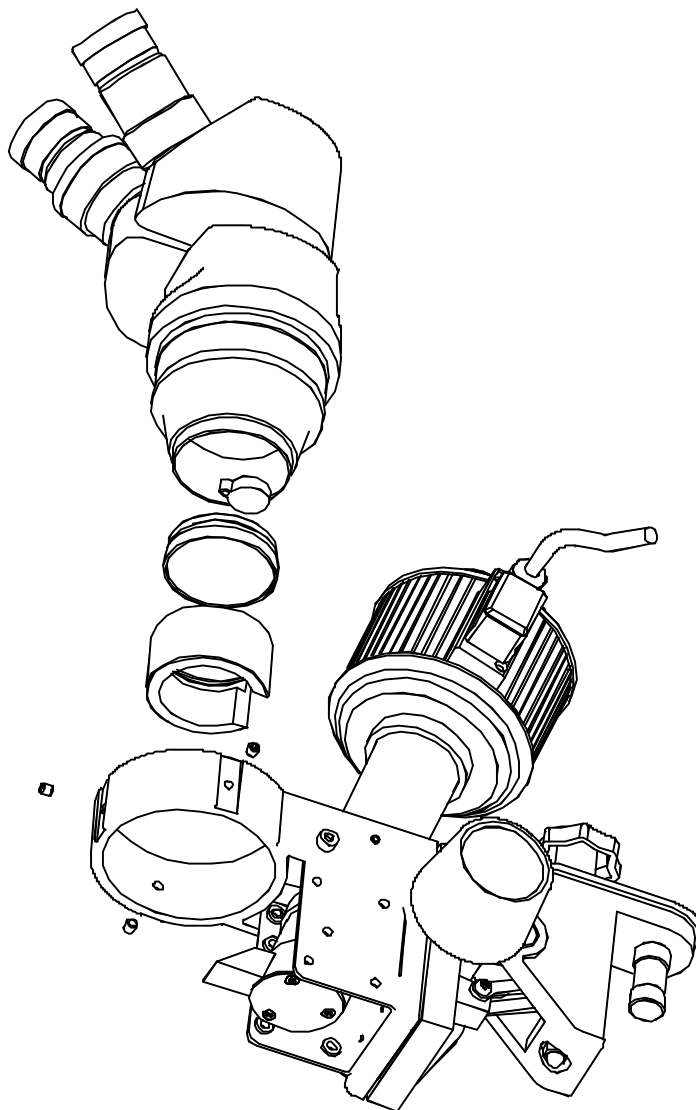


Figure 10: Replacing the mould pellet

## TROUBLESHOOTING

SYMPTOM	CAUSE	REMEDY
No light	No lamp/failed lamp	Replace lamp
	Mains power failure	Use battery back-up
	Fuse blown	Check and replace fuse if necessary
		Check mains voltage setting 110/220V
Fuse repeatedly blows	Incorrect voltage setting	Check mains voltage setting 110/220V
	Mains power fault	Check mains wiring, do not use microscope if wired incorrectly. Use battery backup
Low light level	Intensity turned down	Increase intensity
	Aged lamp	Replace lamp
	Dirty optics	Clean eyepieces, prism and objective lens
	Battery power low	Recharge/replace battery
	Incorrect lamp	Replace with lamp of correct specification
	Lamphouse off-centre	Re-centre lamphouse on image centre (use adjusting grub screws)
Microscope falls under own weight	Insufficient focus friction	Increase focus friction
Blurred vision	Dirty optics	Clean eyepieces, prism and objective lens
	Mouldy optics	Carefully clean mould from optical elements and replace mould protection pellet
	Refractive error	Focus right eye first through RHS eyepiece then adjust LHS eyepiece to compensate for error
Microscope unsteady	Unstable surface	Stiffen mounting surface with a plate
	Knobs loose	Tighten adjustment knobs as necessary



## SPECIFICATIONS

### OPTICAL HEAD

#### VIEWING SYSTEM

Binocular, stereoscopic  
(convergence angle 12°)  
Eyepiece tube inclination 45°

#### MAGNIFICATION

Fixed, 6.25 x  
Optional 9.4x eyepieces

#### WORKING DISTANCE

Lamphouse prism to object distance 165 mm

#### FIELD OF VIEW

44mm

#### REFRACTIVE ERROR

Adjustment +5° to -5°, left eyepiece

#### FOCUSING

Range ± 25mm  
Control knobs removable for sterilisation

#### INTERPUPILLARY DISTANCE

Adjustable for PD range approximately 50 to 80mm

### ILLUMINATION

#### ALIGNMENT

Coaxial with viewing system

#### LAMP

12V 50W quartz-halogen lamp

#### FILTERS

Internal ultraviolet and infrared filters

#### AVERAGE LAMP LIFE

Medium intensity: 50 hours

#### ILLUMINATION

60,000 Lux approximately at maximum setting.

#### LAMP CHANGE

Plug in for fast change, no screws

### POWER SUPPLY

#### MAINS POWER OUTPUT

220-240V/0.2A; 110-120V/0.4A  
12V 50W

#### INTENSITY CONTROL

Continuous

#### EARTHING

via earth lead on mains cable (green/yellow)

#### FUSE

250V/3A 20x5 mm

#### CABLES

*Mains:*

Length 5 metres

*Battery:*

Length 5 metres

**MOUNTING SYSTEM**

HEAD TILT

+5, to -45,

VERTICAL TENSION

Adjustable gas spring to set lifting force

DIMENSIONS

Vertical pillar to head optical axis

max 840 mm (33")

Pantograph arm vertical range 320 mm (13")

MATERIALS

No ferrous metals to rust or corrode

